TRACOE medical GmbH

Reichelsheimer Straße 1 / 3, 55268 Nieder-Olm / Germany

Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200

info@tracoe.com, www.tracoe.com

Instructions for Use for TRACOE® Paraffin Oil (liquid paraffin)

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Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200

info@tracoe.com, www.tracoe.com

Instructions for Use for TRACOE Paraffin Oil (liquid paraffin) to improve sliding of TRACOE *comfort* Tracheostomy Tubes

REF 902

These instructions are for physicians, nursing staff and patients.
They only apply to TRACOE liquid paraffin.

Caution: TRACOE liquid paraffin is only intended for use with
TRACOE *comfort* tracheostomy tubes.

1. Contents

The package contains 25 ml of pharmaceutical-quality liquid paraffin (DAB, Ph Eur, BP, NF, USP), supplied in a pharmaceutical dropper bottle.

It prolongs the life of the TRACOE *comfort* tubes by permitting the inner cannula to slide in the outer cannula more easily.

2. Application

TRACOE paraffin oil is applied in order to permit the inner cannula to slide more easily in the outer cannula of TRACOE *comfort* tracheostomy tubes.

After the tracheostomy tube has been cleaned and dried, a small amount (approx. 1 drop) of the REF 902 Liquid Paraffin is applied to the outside of the inner cannula and distributed with a fluff-free cloth until a thin covering has been achieved. Then the inner cannula is inserted into the outer cannula and pulled back

out and pushed back in several times in order to further distribute the liquid paraffin.

Caution: Make sure that the applied amount of the liquid paraffin is as small as possible. Never apply the oil to an extent that the liquid paraffin escapes between the inner and outer cannula and enters the trachea. There is a possibility, albeit small, that permanent overdosing may result in oil pneumonia.

Please do not replace the TRACOE REF 902 liquid paraffin by alternative products because other oils, for example stoma oil, might glue the inner cannula to the outer cannula.

3. Storage

The liquid paraffin is chemically inert and will not impair the TRACOE *comfort* tracheostomy tubes in any way. The products must be stored in a cool and dry place. Temporary storage for ≤ 30 days at maximum $+50$ °C is possible. Protect the product from sunlight and extreme cold.

4. Returns

Returns of used products are only accepted if they are accompanied by a completed decontamination certificate. You can obtain the required form either directly from TRACOE medical or from the website (www.tracoe.com).

5. General Terms and Conditions

The sale, delivery and return of all TRACOE products are carried out exclusively according to the General Terms and Conditions, which can be obtained directly from TRACOE medical GmbH or from the website (www.tracoe.com).

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