


Stoma Sizing Guide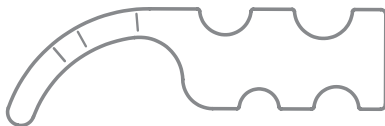


Provox LaryTube / Provox Life LaryTube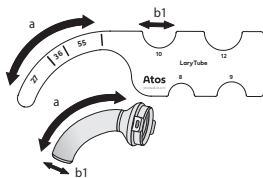

1.

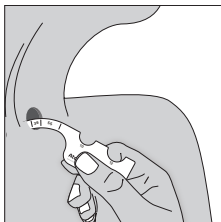
2.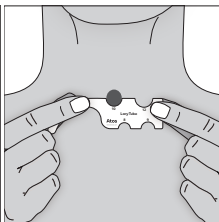


3.


Provox LaryButton / Provox Life LaryButton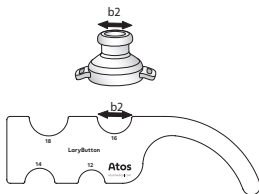

4.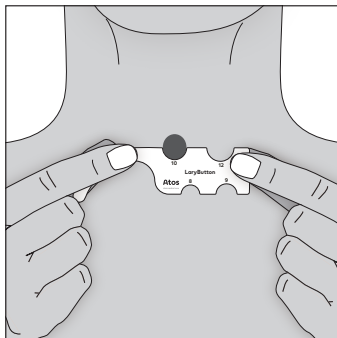

5.

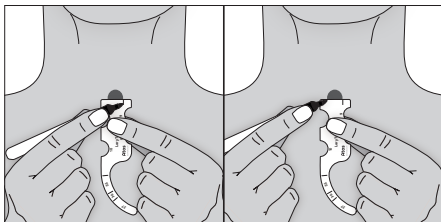6.

Prescription information

CAUTION: United States Federal law restricts this device to sale, distribution and use by or on order of a physician or a licensed practitioner. The availability of this product without prescription outside the United States may vary from country to country.

Disclaimer

Atos Medical offers no warranty – neither expressed nor implied – to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Contents

EN - ENGLISH.....	8
DE - DEUTSCH.....	10
NL - NEDERLANDS.....	13
FR - FRANÇAIS.....	15
IT - ITALIANO	18
ES - ESPAÑOL.....	21
PT - PORTUGUÊS.....	24
SV - SVENSKA	27
DA - DANSK.....	29
NO - NORSK.....	31
FI - SUOMI.....	33
IS - ÍSLENSKA	36
ET - EESTI.....	38
LV - LATVISKI.....	40
LT - LIETUVIŲ KALBA.	42
CS - CESTY	44
HU - MAGYAR	46
SK - SLOVENČINA.....	48

SL - SLOVENŠČINA	51
PL - POLSKI	53
RO - ROMÂNĂ	56
HR - HRVATSKI	59
SR - SRPSKI	61
EL - ΕΛΛΗΝΙΚΑ	63
TR - TÜRKÇE	66
HY - ՀԱՅԵՐԵՆ	69
RU - РУССКИЙ	72
ID - BAHASA INDONESIA	75
MS - BAHASA MELAYU	77
JA - 日本語	80
KO - 한국어	82
ZHTW - 繁體中文	84
ZHCN - 简体中文	86
TH - ไทย	88
VI - TIẾNG VIỆT	90
HE - עברית	93
AR - العربية	95
FA - فارسی	97

1. Intended Use

Stoma Sizing Guide is a single use device intended to help the prescribing clinician determine which size of LaryTube or LaryButton in the Provox and Provox Life range respectively should be prescribed to the patient. Stoma Sizing Guide is intended to be used by a prescribing clinician who has read the IFU for Provox and Provox Life LaryTube and LaryButton respectively.

Stoma Sizing Guide can also be used by patients to monitor the stoma size.

2. Warnings

- Do not reuse the device on another patient, as this may cause cross-contamination.
- Do not alter, modify or use a damaged device, as this may cause aspiration of parts, paper and ink in contact with mucosa, which may cause allergic or toxic reaction.
- Do not use a dirty or contaminated device, as this may cause infection.
- Do not use force during insertion as tissue damage, bleeding or irritation and aspiration of parts and/or accidental dislodgement of the voice prosthesis may occur.
- Do not drop/let go of the device when measuring length, may lead to accidental aspiration.

3. Precautions

- Ensure proper evaluation of the stoma, a too large prescribed button/tube may cause irritation or injury, and possibly bleeding or hemorrhage. A too large button/tube can also exert pressure on the voice prosthesis or

on the tracheal wall, which can cause tissue damage or necrosis. A too small size of button/tube may cause the stoma to shrink.

- Ensure the patient/caregiver has been trained how to use the device.

4. Instructions for Use

Healthcare professionals refer to figures 1-5.

5. Patient/caregiver training

Patients or caregivers may use the Stoma Sizing Guide to monitor the stoma size after they have been trained by a healthcare professional. The patient / caregiver should demonstrate the ability to understand and consistently follow the Patient / Caregiver Stoma Sizing Guide Instructions for Use without clinician supervision. Patients / caregivers refer to figure 6.

6. Disposal

Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

7. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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Atos and the Atos logo are trademarks of Coloplast A/S.
© 2025 Coloplast A/S. All rights reserved.
Atos Medical AB