

## Cleaning

1. Rinse the SilcoSoft tube under lukewarm potable water (40°C) for at least 3 minutes
2. For incrustations, soak in water at temperatures of up to 65°C up to 30 minutes
3. Repeat the rinsing procedure (see above)
4. Allow to air-dry

## Reprocessing

If necessary, a disinfection in a pot of boiling water (100°C) can be done for 3 minutes (typically used in homecare). Remove the pot from the heat source. The tube should remain in covered pot of water for 30 minutes, then allow to air-dry.

For sterilization, the tracheostomy tubes are also autoclavable (moist heat/steam).

SilcoSoft is a single patient use product, intended to be used for 29 days. Reprocessing up to 7 times within 29 days.

### Head office:

Atos Medical AB  
Hyllie Boulevard 17  
SE-215 32 Malmö, Sweden

Tel: +46 (0) 415 198 00  
Email: [info@atosmedical.com](mailto:info@atosmedical.com)

### Manufacturer:

Tracoe Medical GmbH  
Reichelsheimer Straße 1/3  
55268 Nieder-Olm, Germany

The information contained in this material does not replace the Instructions for Use of the products demonstrated. Always read the Instructions for use before starting to use any of the products. For Instructions for use, please visit [www.atosmedical.com](http://www.atosmedical.com).

© Coloplast A/S. All rights reserved. Atos and the Atos Medical logo are trademarks of Coloplast A/S.