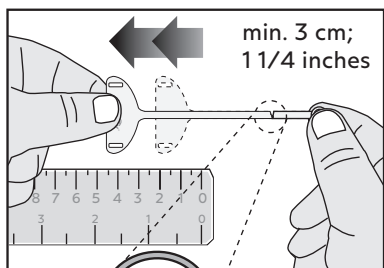
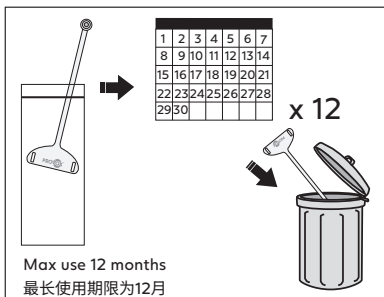
PROVOX[®]

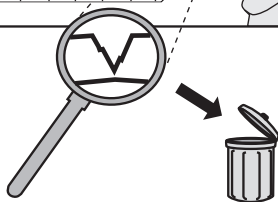
Vega[™] Plug

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
1.


The original length is 1 1/4 inches (about 7 centimeters). Pull the strap at least 3 centimeters. If there is any breakage, please discard it.

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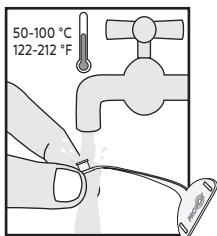
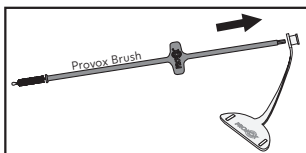
2. Rinse the insertion end of the device.
冲洗塞子的插入端

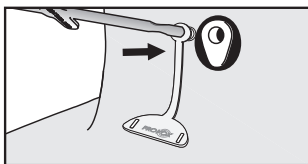
3. Secure Provox Vega Plug by using Provox Brush.
用Provox Brush清洁刷将塞子固定

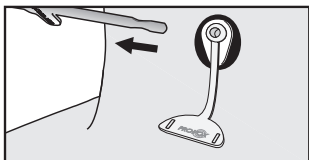4. Insert Provox Vega Plug into the lumen
of the prosthesis
正对辅助发音管并插入小孔

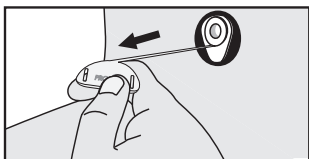
5. Remove Provox Brush.
移除Provox Brush清洁刷

6. Pull the end of the plug and slightly to
remove the Plug.
稍微用力拉塞子在体外的一端可将塞子取下


2 / 250 ml (8,5 fl oz)

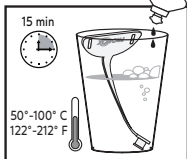Dishsoap


7. Two drops of mild soap adding to 250ml water for soaking and cleaning.

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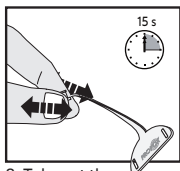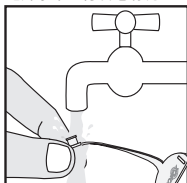
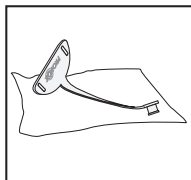
9. Rinse the device in water
流动的水冲洗

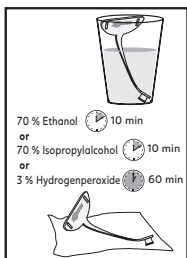10. Let the device dry
晾干

11. Soak in 70% ethanol for 10 minutes, or soak in 70% isopropanol for 10 minutes, or soak in 3% hydrogen peroxide solution for 60 minutes.

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Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Due to local Chinese requirements, the text in the Intended use paragraph and the Overall description and product composition paragraph are not translated verbatim.

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1. Descriptive Information

1.1 Intended use

Provox Vega Plug is intended to seal the inner lumen of Provox Vega voice prosthesis and therefore stop leakage of both air and fluids through the voice prosthesis.

1.2 Overall description and product

composition (结构及组成)

Provox Voice Prosthesis is composed of Silicone, Polyvinylidene Fluoride (PVDF) and Silicone adhesive. The insertion system in the Provox Vega Puncture Set is composed of MethylMethacrylate Acrylonitrile Butadiene Styrene (MABS), Stainless steel, Polyamide (PA66) and epoxy glue, Thermoplastic Styrene-Ethylene/Butylene-Styrene (TPS-SEBS), polypropylene (PP), Polyvinylidene Fluoride (PVDF).

The insertion system in the Provox Vega and Provox Vega XtraSeal is composed of Polypropylene (PP), colorant, and Silicone oil.

Provox Flush in the Provox Accessories is composed of Silicone, Polypropylene (PP) and blue masterbatch; Provox Vega Plug in the Provox Accessories is made of Silicone.

Provox Voice Prosthesis and its set are sterilized by ethylene oxide.

The product is for single use.

The Provox accessories are non-sterile products.

The shelf life of package sterilization is 5 years.

1.3 CONTRAINDICATIONS

None.

1.4 WARNINGS

None.

1.5 Description of the device

Provox Vega Plug is a first-aid tool for temporarily stopping leakage through a Provox Vega voice prosthesis. The device is inserted into the opening of the Provox Vega voice prosthesis and hence blocking any leakage through the valve.

1.6 Precaution

Provox Vega Plug is intended for single patient use. Reuse in another patient may cause cross contamination.

2. Instructions for use

2.1 Operation instruction

Check device integrity prior to each use by pulling the strap see fig. 1.

For instructions on how to use the product, please see fig. 2-6.

2.2 Cleaning and disinfection

Clean the Provox Vega Plug after each use, or if it is changed during use according to Fig. 7-10.

Fig. 8; Rub the device between your fingers for 15 seconds.

DO NOT use any water other than drinking water to clean and rinse the device.

Disinfect the Provox Vega Plug at least once a day if it is removed (Fig. 11) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

CAUTION: DO NOT clean or disinfect by any other method as this might cause product damage and injury.

If the Provox Vega Plug looks dirty or has air dried in an area with a risk of contamination the device should be both cleaned and disinfected before use.

A risk of contamination could exist if the device has been dropped on the floor or if it has been in contact with a pet, someone with a respiratory infection, or any other gross contamination.

During hospitalization it is important to both clean and disinfect the Provox Vega Plug after use but also before use since there is an increased risk of infection and contamination. In a hospital it is better to use sterile water for cleaning and rinsing, rather than drinking water.

2.3 Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

3. Product model

Model	Specification
8119-18	17Fr
8129-18	20Fr
8139-18	22.5Fr

4. Storage conditions

Store the product dry and away from sunlight at room temperature. Excursions permitted between 2°C - 42°C.

5. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

6. Legal Agent and After-sales service information

Coloplast (China) Medical Devices Ltd.
Address: Unit1301-1306, 13th Floor, Building 1, No.5
Lido Huayuan Road, Chaoyang District, Beijing
Agent contact information:
Tel: 010-5920 1888
Fax: 010-5920 1898
Coloplast Customer Service Hotline
Hotline: 400 700 7668
Website: www.coloplast.com.cn

7. Production date and validity

Please see the label for the production date; the shelf life of product is 5 years.

8. Registration certificate number of medical devices

Certificate number of medical devices: 国械注进
20223130609

9. Technical requirements for medical devices

Technical requirement number for medical devices:
国械注进20223130609

10. Edition of Instruction for Use

Article number: 11704, Edition date: 2026-03-19.

11. Registration and Manufacturer company

Atos Medical AB

Registration & Manufacturer location: Kraftgatan 8,
SE-242 35 Hörby, Sweden

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Atos and the Atos logo are trademarks of Coloplast A/S.
© 2026 Coloplast A/S. All rights reserved.