FREEVENT®

Dressing

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Freevent[®] is a registered trademark owned by Atos Medical AB, Sweden.
For information about protective rights (e.g. patents), please refer to our web page www.atosmedical.com/patents.

Contents

EN - ENGLISH.....	4
DE - DEUTSCH	5
NL - NEDERLANDS.....	6
FR - FRANÇAIS.....	7
IT - ITALIANO	8
ES - ESPAÑOL	9
PT - PORTUGUÊS	10
SV - SVENSKA.....	11
DA - DANSK	12
FI - SUOMI	13
HR - HRVATSKI	14
TR - TÜRKÇE.....	15

1. Intended use

Freevent Dressings are single use tracheal dressings that provide protection between the tracheal cannula and the skin and absorb secretions.

2. Operating instructions

Place Freevent Dressings between the tracheal cannula and the skin. When using Dressings without slit, the tracheal cannula is inserted through the opening of the dressing. The tracheal cannula is then inserted in the tracheostoma and fixed with a tracheal cannula holder.

3. WARNINGS

- When changing the dressing be very careful in order to prevent accidental de-cannulation.

4. PRECAUTIONS

- The device may irritate the skin. Stop using the device if skin irritation develops and consult your clinician.
- The dressing should be changed at least every 24 hours or more often if needed to prevent skin irritation.
- Do not modify the dressing by e.g. cutting.

5. Device performance

For hygienic reasons, the Freevent Dressing should be exchanged when needed, at least with every cannula change, or every 24 hours.

6. Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

7. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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