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 Packaging unit

Purpose:

The **PRIMA-SMOOTH** tracheostoma levelling disc seals an uneven tracheostomy. It is used in conjunction with a tracheostomy plaster.

Description:

The **PRIMA-SMOOTH** tracheostoma levelling disc is generally used in conjunction with a tracheostomy plaster. Prior to use, the protective film is removed from the adhesive side (illustration 1). The **PRIMA-SMOOTH** is positioned with the adhesive side on the tracheostomy (illustration 2). Depending on the degree of unevenness, it may be necessary to attach a second **PRIMA-SMOOTH** tracheostoma levelling disc to the first, in order to seal the tracheostomy completely. The plaster is then placed over the tracheostoma levelling disc ensuring the opening remains clear and unobstructed (illustration 3). The **PRIMA-SMOOTH** tracheostoma levelling disc is a disposable product for use by one patient only and must be changed daily for reasons of hygiene. An earlier changing is indicated if the tracheostoma levelling disc becomes contaminated with secretions.

Cleaning:

For reasons of hygiene the **PRIMA-SMOOTH** tracheostoma levelling disc may not be re-processed.

Safety Information:

Risk of infection follows if the **PRIMA-SMOOTH** tracheostoma levelling disc is re-used or its use exceeds the prescribed duration. No allergic reactions have so far been reported.

Product Disposal:

Disposal of the product – in a germ-free condition – may be carried out only in accordance with current national regulations for waste materials.

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