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Explanation of symbols:

 Packaging Unit

 **PRIMA-STOM II** oil may be used up to 1 month after opening.

Purpose:

PRIMA-STOM II oil is for care of **PRIMA-STOM** tracheal cannulae and other plastic cannulae. Also, insertion of the inner cannula into the outer cannula is facilitated due to the lubricant function of **PRIMA-STOM II** oil.

Description:

After every cleaning of the tracheal cannula, a few drops of **PRIMA-STOM II** oil can be rubbed onto the inner cannulae. This leaves plastic cannulae pliable and facilitates the removal of the inner cannula.

To improve dosing, **PRIMA-STOM II** oil is supplied in a dropper bottle with a screw closure and can be distributed over the cannula using a compress.

PRIMA-STOM II oil must not be used on silicone cannulae.

Safety Information:

To this day no cases of allergic reaction to **PRIMA-STOM II** oil have been reported. There is a risk of infection if the recommended period of use or the expiration date is exceeded.

Product disposal:

Disposal of products must be carried out in compliance with current national regulations applying to waste products.

Caution: Warranty, guarantee and liability claims against the manufacturer will be nullified if any changes are made to the product by users or if the product is used outside of the scope of applications described in these instructions.

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