# PRIM-AIR PED II/III

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 Packaging unit

**Purpose:**

**PRIM-AIR PED II/III** HMEs (heat-moisture exchangers) humidify, filter and warm the inhaled air.

**Description:**

Remove the filter from its packing and attach it to the universal connector. **PRIM-AIR PED II/III** HMEs were specially designed for the **PRIMA-STOM II PED** paediatric tracheal tube series. They can, however, also be used in treatment of adults. **PRIM-AIR PED II/III** HMEs are therefore suitable for all tracheal tubes with the HEIMOMED universal adapter. By using a safety-cap or speech-valve the HME protects against coughing up. If using a speech-valve or safety-cap, care must be taken to ensure correct positioning of the retention filament between the universal connector and the safety-cap, to prevent aspiration of the HME (see illustration).

**PRIM-AIR PED II/III** HMEs are single-use products. If the HME is clogged with secretion it must be replaced. This mainly occurs during the 2-3 day familiarisation period. Later it can be worn for up to a maximum of 24 hours.

**Indication:**

Patients with spontaneous respiration via the tracheotomy, wearers of tracheal canulas in the hospital or a domestic environment.

**Contraindication:**

- The **PRIM-AIR PED II/III** should be removed if breathing resistance is too high. Increased dead space can lead to insufficient moisturisation.
- Use in patients with severely depleted body fluids (exsiccosis) is not permitted, nor in case of excessive secretion in lungs and airways as well as pronounced air leakage (expired air does not flow through the HME).

**Technical data PRIM-AIR PED II:**

Interior volume: 1,99 cm<sup>3</sup>  
Weight: 1,2 g  
Length: 15 mm  
Diameter: 13 mm

**PRIM-AIR PED III:**

Interior volume: 1,99 cm<sup>3</sup>  
Weight: 1,8 g  
Length: 15 mm  
Diameter: 13 mm

**Safety Information:**

Should the HME become blocked or show signs of other malfunction it must be removed and disposed of immediately. It should not be used a second time, nor should it be cleaned. The use of disinfecting agents can render the HME useless. Sterilisation for re-use can impair the function of the HME and is therefore not permitted. Inattention to the correct instructions for use, especially wrong positioning of the retention filament, can result in aspiration of the unit. Infection can be caused by prolonged use or unauthorised use. The **PRIM-AIR PED II/III** may not be used in combination with a hot water humidifier or nebuliser. Water must not enter the **PRIM-AIR PED II/III** as this may induce pronounced breathing resistance.

**Product Disposal:**

Disposal of HME in germ-free condition may be carried out only in accordance with current national regulations for waste materials. Combustion of the product releases hydrocarbons, water and carbon oxides.

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